YESOS · STONES · PLÂTRES · GESSI

MUFLA-GIPS

HOJA DE INSTRUCCIONES	ES	3
WORKING INSTRUCTIONS	EN	4
NOTICE D'EMPLOI	FR	5
ISTRUZIONI PER L'USO	IT	6
INFORMACIÓN ADICIONAL ADDITIONAL INFORMATION INFORMATIONS COMPLÉMENTAIRES INFORMAZIONI SUPPLEMENTARI		2 & 7

Consulte nuestra web
Visit our website

www.protechno.com

CARACTERÍSTICAS FÍSICAS · PHYSICAL CHARACTERISTICS
CARACTÉRISTIQUES PHYSIQUES · CARATTERISTICHE FISICHE

100 g / 35-40 ml

Proporción polvo/agua desmineralizada
 Powder/ demineralised water ratio
 Dosage poudre/eau déminéralisée
 Proporzioni polvere/acqua demineralizzata

20°C - 23°C

Temperatura de trabajo
 Working temperature
 Température de travail
 Temperatura di lavorazione

60''

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MUFLA-GIPS

DESCRIPTION

Fluid plaster for flasking. Type II, ISO 6873.

USES

Very fluid plaster to be poured into the upper half of the flask without vibration. Recommended for all kind of flasks to make the upper half of the flask.

ADVANTAGE

S

- Once set, the obtained surface is exceptionally smooth, which greatly reduces air pockets and bubbles in the acrylics.
- Free of bubbles even without vibration and with manual mixing.
- Does not adhere to acrylic denture or teeth.
- Natural white colour.

INSTRUCTIONS OF USE

Add the powder to the water following the recommended ratio.

When this mixing time has finished, the mix has a great fluid consistency, flowing easily into the flask even without vibration.

CAUTIONS

1.- USE CAUTIONS

Use always-clean instruments. Do not mix the product in a container where investments have been mixed. Use always demineralised water to do the mix. The water from the public mains can affect the correct setting process of the gypsum, making it faster or slower.

2.- SAFETY CAUTIONS

Do not inhale the powder. It's recommended the use of a protective mask. Keep away from children. Handle the product in a well-ventilated place. The handling of this product has to be done only by qualified people. In the event of ingestion, consult a doctor. For further information, see the safety data sheet for the product.

EXPIRY AND STORAGE

EXPIRY

The stability of the product is 24 months respecting the storage conditions. (Note: The expiry date is considered with the product in his original packing and before been opened).

STORAGE

Keep the packages tightly closed in a dry place away from moisture. After each use the package must be closed to prevent the moisture absorption.

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EN

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet of MUFLA-GIPS. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review.

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