



# DURGUIX

MODEL STONE / **STONES**



## HIGH QUALITY STONE

- STONE FOR GENERAL LABORATORY USES
- FOR THE FABRICATION OF MODELS IN ACRYLIC AND CR-CO TECHNIQUE,  
OPPOSITE MODELS AND IN THE FORMATION OF  
IMPRESSION TRAYS

STONE TYPE 3 • ISO 6873

# DURGUIX

**HARD NATURAL STONE. TYPE 3, HIGH QUALITY.**

## ADVANTAGES

- **Thixotropic** consistency
- Hard and even surfaces
- Good definition of details

## USE INSTRUCTIONS

|                      |                  |
|----------------------|------------------|
| Powder/water ratio:  | 100 g / 28-30 ml |
| Working temperature: | 20-23°C          |
| Mixing: Vacuum:      | 30 sec.          |
| Manual:              | 60 sec.          |

## PHYSICAL CHARACTERISTICS

|                                    |                        |
|------------------------------------|------------------------|
| Working time:                      | 5 min.                 |
| Initial setting:                   | 9 min.                 |
| (Vicat test according to ISO 6873) |                        |
| Final setting:                     | 12 min.                |
| (Vicat test according to ISO 6873) |                        |
| Setting time:                      | 30 min.                |
| Setting expansion (2 h.):          | 0.16 %                 |
| Compressive strength (1 h.):       | 32 N / mm <sup>2</sup> |

## SALES FORMAT



### Pail 20 kg

| Pink     | Yellow   | Blue     |
|----------|----------|----------|
| 1140-200 | 1140-201 | 1140-202 |

### Paper bag 25 kg

| Pink     | Yellow   | Blue     |
|----------|----------|----------|
| 1140-260 | 1140-261 | 1140-262 |

### Box 25 kg

| Pink     | Yellow   | Blue     |
|----------|----------|----------|
| 1140-280 | 1140-281 | 1140-282 |



## RELATED PRODUCTS


### TENSIO-SIL

Tension-reducing fluid for alginate and silicone. Bubble-free models.



### QUICK-ROCK

Mounting stone.

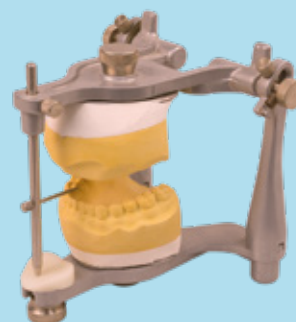

**protechno**

ADVANCED DENTAL PRODUCTS



Tel. +34 972 526 169

Polígono Empordà Internacional  
17469 . VILAMALLA (Girona) . Spain

**protechno@protechno.com**

**www.protechno.com**