DISOL-GIPS

Descripción

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DISOL-GIPS

Description

Powerful gypsum and alginate removing liquid

Uses

To remove gypsum from dentures, and also to remove gypsum and alginate from impression trays, spatulas, articulators, etc

DISOL-GIPS liquid can be used to clean metallic and plastic trays **except the aluminium** ones. It can also be used to clean mixing bowls and spatulas.

It removes gypsum from acrylic dentures even from the difficult access spaces without changing the colour of the prosthesis

Advantages

- Ready to use. Dissolving it's not necessary
- Fast reaction: bubble formation
- It can be used in ultrasonic bath to accelerate its action

Application

Introduce the object to clean in a suitable glass or plastic container full of DISOL-GIPS


Gypsum removal

To clean the rests in dentures, spatulas or mixing bowls, submerge the product which has to be cleaned for about 1 hour in DISOL-GIPS

Once the product is clean, take it out and rinse it with water.

Alginate removal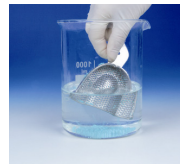

To remove rests of alginate from impression trays it's better to let them in DISOL-GIPS overnight

The alginate rests get softer providing an easier Cleaning with a soft brush or rinsing it with water

For both applications the same liquid can be reused for 5-6 times (according to the amount of gypsum / alginate that has to be removed). The use can be extended although it loses progressively its cleaning power

Storage and stability

The stability of the product is 3 years respecting the storage conditions. (Note: The expiry date is considered with the product in its original packing and before been opened).

Storage: Keep the product tightly closed in its original package and in a dry cool place.

Packing

1191-300 DISOL-GIPS 3 l

Additional information

The information given in this sheet is based on our current knowledge. Its only given for information purposes and is not intended to be a warranty for the product.

protechno

Polígono Empordà Internacional
17469 VILAMALLA, GIRONA , SPAIN
Tel.: (34) 972 52 61 69
protechno@protechno.com
www.protechno.com