SPACER-LAC

Descripción

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Utilización

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Die varnish for painting stumps

Uses

Varnish for painting stumps, serving as separator for the cement, at the same time sealing the stone surface.

Advantages:

- Shows the moment when it's dry changing its color from orange to red
- Perfect results for the inlay and onlay technique because of its fluid consistence
- Clear, the marked lines rest visible
- Without coloring pigments, it's no necessary to shake the bottle before use.

Use instructions

Apply the varnish 1 mm above the cervical limit

Then a layer of 7 microns is obtained over the model

Let dry until the change of color

Safety cautions

High flammable product

Work in well ventilated places

Keep away from naked flames, hot items and sparks

Do not eat, drink or smoke while using

Avoid contact with skin or eyes and the inhalation of gases

In the event of ingestion, consult a doctor

For further information, see the safety data sheet of the product

Storage

The stability of the product is 2 years respecting the storage conditions. (Note: The expiry date is considered with the product in his original packing and before been opened).

Storage: Keep the product tightly closed in its original package and in a dry cool place.

Packing

3210-252 Bottle 25 ml color red

Thinner
3220-250 Bottle 25 ml

Additional information

The information given in this sheet is based on our level of knowledge at the moment at the date of printing and is not intended to be a warranty for the product.

protechno

Polígono Empordà Internacional

17469 VILAMALLA, GIRONA ,

SPAIN Tel.: (34) 972 52 61 69

protechno@protechno.com

www.protechno.com