SCAN-LAC

Descripción

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SCAN-LAC

Description

Anti-reflective paint for CAD/CAM scanning.

Uses

Paint to eliminate scanning defects in shiny areas.
Especially recommended for metal parts.

Advantages:

- Provides a very thin, completely opaque and reflection-free layer.
- Easily removable with water only.
- Direct application with a brush, improving control over the application area.

Recommended uses

- For metal abutments in cemented implant prosthetics.
- For metal positioners in screw-retained implant prosthetics.

Other uses:

Non-scannable wax patterns.

Gum silicone.

Non-scannable or reflective gypsum molds.

Instructions for use

Vigorously shake the container until a uniform product is observed.

The consistency of the product is quite liquid. **Due to its composition, it can easily evaporate if the container remains open.**

We recommend pouring a small amount of product into the supplied cup to keep the bottle open for the shortest possible time.

A bottle of liquid thinner is included to adjust the product density in case it is too thick, which would create a layer that is too thick.

Paint the area with reflections to be scanned using the brush, creating a thin layer.

Allow a few seconds to dry, and the piece is ready to be scanned.

Any remaining residue can be easily removed with water.

Safety cautions

Highly flammable product.

Work in well-ventilated areas.

Keep away from flames, hot surfaces, and sparks.

Do not eat, drink, or smoke during use.

Avoid contact with eyes, skin, and inhalation of fumes.

In case of ingestion, consult a physician.

For more information, refer to the product safety data sheet.

Storage and expiration

The product's stability is 2 years when stored under proper conditions. The expiration date refers to the product in its original packaging and prior to opening.

Storage: Keep the product tightly closed in its original packaging in a cool, dry place.

Packaging

0610-150: Kit of 2 bottles, 15 ml each (paint + thinner)

Includes brush and dosing cup.

Additional information

The information given in this sheet is based on our level of knowledge at the moment at the date of printing and is not intended to be a warranty for the product.

protechno

Polígono Empordà Internacional

17469 VILAMALLA, GIRONA, SPAIN

Tel.: (+34) 972 52 61 69

protechno@protechno.com

www.protechno.com