



DISOL-GIPS

ACCESSORIES / **STONES**



STRONG GYPSUM AND ALGINATE REMOVER



DISOL-GIPS IS DESIGNED TO REMOVE GYPSUM FROM DENTURES AND ALSO TO REMOVE GYPSUM AND ALGINATE FROM IMPRESSION TRAYS, SPATULAS, ARTICULATORS , ETC.


protechno
ADVANCED DENTAL PRODUCTS

DISOL-GIPS

STRONG GYPSUM AND ALGINATE REMOVER.

ADVANTAGES

- Fast reaction: bubble formation
- It can be used in ultrasonic bath to accelerate its action
- Ready to use. Dissolving is not necessary
- Do not damage the prosthesis

USE INSTRUCTIONS

Introduce the object to clean in a suitable glass or plastic container full of **DISOL-GIPS**.

To remove the stone rests, leave the product work for hours. To remove alginate, better overnight. Rinse with water.

DISOL-GIPS liquid can be used to clean metallic and plastic trays except the aluminium ones. It can also be used to clean mixing bowls and spatulas.

It removes gypsum from acrylic dentures even from the difficult access spaces without changing the colour of the prosthesis.

STORAGE AND STABILITY

The stability of the product is 3 years respecting the storage conditions.

SALES FORMAT



1191-300 DISOL-GIPS

3.000 ml


- Ready to use



REMOVING
STONE FROM
DENTURES

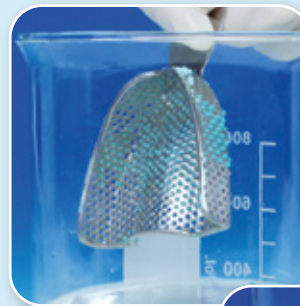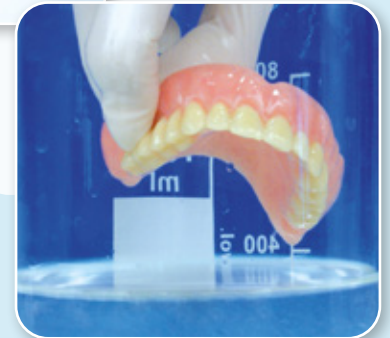

CLEANING
TRAYS AND
ALGINATE
REMAINS

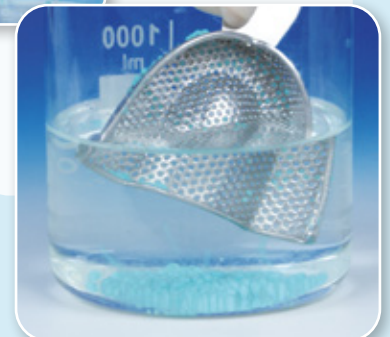

protechno

ADVANCED DENTAL PRODUCTS



Tel. +34 972 526 169

Polígono Empordà Internacional
17469 . VILAMALLA (Girona) . Spain

protechno@protechno.com

www.protechno.com