



KIMBERLIT

STUMPS & MODELS / STONES



TIME IS MONEY: MODELS IN 20 MINUTES

- GET THE HIGHEST QUALITY WITH INCREASE PRODUCTIVITY
- IDEAL FOR ALL TYPES OF FIXED PROSTHESES, STUMPS AND MASTER MODELS
- CAD / CAM COMPATIBLE

STONE TYPE 4 • ISO 6873



protechno

ADVANCED DENTAL PRODUCTS

KIMBERLIT

NATURAL EXTRA-HARD HIGH DENSITY DIE-STONE.

ADVANTAGES

- **Accurate demoulding in only 20 min.**
- Stump cut without breaking
- Thixotropic
- Extra high resistance

USE INSTRUCTIONS

Powder/water ratio:	100 g. / 20 ml.
Working temperature:	20-23°C
Mixing: Mechanical under vacuum:	30 sec.
Manual:	60 sec.

PHYSICAL CHARACTERISTICS

Working time:	7 min.
Initial setting:	9 min.
(Vicat test according to ISO 6873)	
Final setting:	13 min.
(Vicat test according to ISO 6873)	
Setting expansion (2 h.):	0.08 %
Hardness (24 h.):	247 N / mm²
Compressive strength (1 h.):	58 N / mm²

SALES FORMAT

Eco-Pack 12 Kg (6 x 2 Kg)

Golden	White
1105-120	1105-121

Pail 18 Kg

Golden	White
1105-180	1105-181

Test Pack 2 Kg (1 x 2 Kg)

Golden	White
1105-K20	1105-K21



ASSOCIATED PRODUCTS

ACCU-PIN

Precision Die System, one-pour technique of stumps and assembling models.



TENSIO-SIL

Tension-reducing fluid for alginate and silicone. Non porous models.



protechno

ADVANCED DENTAL PRODUCTS



Tel. +34 972 526 169

Polígono Empordà Internacional
17469 . VILAMALLA (Girona) . Spain

protechno@protechno.com

www.protechno.com