



RESIN-GIPS PLUS

STUMPS & MODELS / STONES



FLUID STONE WITH RESIN

- MORE CONTENT IN RESIN
- BETTER RESISTANCE TO ABRASION
- ITS GREY COLOUR ALLOWS BETTER FINISHING
- VERY SMOOTH SURFACES. MAXIMUM HARDNESS

TYPE 4 • ISO 6873


protechno
ADVANCED DENTAL PRODUCTS

RESIN-GIPS PLUS

EXTRA HARD RESIN ADDED DIE-STONE.

TYPE 4, HIGH QUALITY.

USE INSTRUCTIONS

Specially designed stone for implants. As well as for veneers, inlays and removable dies.

ADVANTAGES

New improved formula:

- **Higher resin content**
- **Greater resistance to abrasion**

Due to the formulation with resin, you can obtain very hard and even surfaces.

Compatible with all kind of impression materials.

Controlled setting expansion which provides a good dimensional stability.

Easy to cut avoiding the formation of dust in the laboratory

Very fluid gypsum: easy to use.

Exact reproducibility.

Snap setting effect.

WORKING INSTRUCTIONS

Powder/water ratio: 100 g / 20 ml

Working temperature: 20-23°C

Mixing: mechanical under vacuum: 30 sec.

Add the powder to the demineralised water following the recommended ratio.

SALES FORMAT

1108-062 **Eco-pack 6 kg** Colour Grey
(3 x 2 kg Al bag)

1108-K20 **Test Pack 1 kg** Colour Grey
(1 x 2 kg Al bag)



PHYSICAL CHARACTERISTICS

Working time: 7 min.

Initial setting: 8 min.

(Vicat test according to ISO 6873)

Final setting: 10 min.

(Vicat test according to ISO 6873)

Setting time: 30 min.

Setting expansion (2 h.): 0.10 %

Compressive strength (1 h.): 50 MPa



protechno

ADVANCED DENTAL PRODUCTS



Tel. +34 972 526 169

Polígono Empordà Internacional
17469 . VILAMALLA (Girona) . Spain

protechno@protechno.com

www.protechno.com