



## ELIMINATING SCANNING DEFECTS

- PAINT TO ELIMINATE SCANNING DEFECTS IN SHINY AREAS
- ESPECIALLY RECOMMENDED FOR METAL PARTS



# SCAN-LAC

## ANTI-REFLECTIVE PAINT FOR CAD/CAM SCANNING

### APPLICATIONS

- For metal abutments in cemented implant
- prosthetics.
- For metal positioners in screw-retained implant prosthetics

### ADVANTAGES

- Provides a very thin, completely opaque and reflection free layer.
- Easily removable with water only.
- Direct application with a brush, improving control over the application area.

### INSTRUCTIONS FOR USE



The consistency of the product is quite liquid. Due to its composition, it can easily evaporate if the container remains open.

We recommend pouring a small amount of product into the supplied cup to keep the bottle open for the

shortest possible time.

A bottle of liquid thinner is included to adjust the product density in case it is too thick, which would create a layer that is too thick.

Paint the area with reflections to be scanned using the brush, creating a thin layer.

Allow a few seconds to dry, and the piece is ready to be scanned.

Any remaining residue can be easily removed with water.

### STORAGE AND EXPIRATION

The product's stability is 2 years. Keep the product tightly closed in its original packaging in a cool, dry place

### SALES FORMAT

0610-150 15 ml + 15 ml + paintbrush Kit



## RELATED PRODUCTS

**KIMBERLIT** | Extra hard high density die stone.



**protechno**

ADVANCED DENTAL PRODUCTS



Tel. +34 972 526 169

Polígono Empordà Internacional  
17469 . VILAMALLA (Girona) . Spain

[protechno@protechno.com](mailto:protechno@protechno.com)

[www.protechno.com](http://www.protechno.com)