



TENSIO-SIL

SEPARATING



MODELS WITHOUT POROSITY

- TENSION-REDUCING FLUID WHICH PREVENTS POROSITY DUE TO TRAPPED AIR
- FACILITATES THE FLOW OF PLASTER OVER THE IMPRESSION SURFACE, EVEN ON THE SMALLEST DETAILS



protechno

ADVANCED PRODUCTS FOR DENTAL LABS

TENSIO-SIL

TENSION-REDUCING FLUID

USE INSTRUCTIONS

- Plaster casting of silicone, alginate or polyether impressions
- Fabrication of duplicate models on a silicone mould

ADVANTAGES

- Models with a high definition of detail which are free from porosity are obtained
- Water-based: Does not contain alcohol, therefore, does not require drying prior to pouring and does not affect the setting of the plaster
- Practical refillable spray + 1 litre refill bottle

WORKING INSTRUCTIONS

Apply, preferably by spraying or with a brush, evenly over the mould or impression tray.

Remove excess product by gently shaking the tray, without the need for drying.

Pour the plaster with a slight vibration.

STORAGE AND EXPIRY DATE

The product remains stable for two years. Store the product in its original container, firmly closed, and in a cool, dry place.

SALES FORMAT

3260-104	Refill Bottle	1.000 ml
3260-124	Spray	125 ml
3260-M54	TEST PACK Spray	50 ml



ASSOCIATED PRODUCT

KIMBERLIT

Extra-hard high density die stone.



ADVANCED PRODUCTS FOR DENTAL LABS



Tel. +34 972 526 169
Polígono Empordà Internacional
17469 . VILAMALLA (Girona) . Spain
protechno@protechno.com
www.protechno.com