



PROTEC-FIX

PHOTOPOLYMERISABLE / ACRYLIC RESINS



DENTURES FIXED IN 10 MINUTES

- LIGHT CURING GEL FOR DENTURES REPARATION
- HIGH STABILITY
- EASY APPLICATION WITH SYRINGE



protechno

ADVANCED PRODUCTS FOR DENTAL LABS

PROTEC-FIX

PHOTO-POLYMERISABLE CURING RESIN GEL, FOR REPAIRING WORKS.

DESCRIPTION

PROTEC-FIX cures between 360 and 480 nm and can be cured using any light-curing unit working with quoted spectrum.

It is cured without vacuum.

COMPOSITION: Acrylat-/methacrylate resins, additives and pigments.

APPLICATIONS

- Fabrication of anterior immediate prostheses with a single missing tooth
- Saddles relining of partial dentures
- Completion of small-sized saddles of partial or implant dentures
- Repairs caused by cracks or fractures on acrylic dentures or base parts
- Fix of broken off teeth on full or partial dentures
- Correction of bubbles, pores and cracks on acrylic denture bases

ADVANTAGES

- Easy application with a syringe
- Immediately positionally stable
- Quick and exact results without using kneading silicon
- Free of MMA

PROCESSING

Once opened, use the one-use provided cannulas for a precise application.

Close the syringe after use.



1. Apply **PROTEC-FIX Bonder** before applying the resin with a brush in each acrylic part (or tooth).



2. Polymerize approximately 3 min.



3. Then apply **PROTEC-FIX** with the syringe. Apply an even coat of material (1-2 mm).



4. Polymerize approximately 3 min.

To apply material after curing, do not remove the inhibition layer on the surface. It guarantees an optimum bonding and avoids any stress to the resin.

Upon completion remove the inhibition layers from the surface with rotary instruments or isopropanol.

FINISHING AND POLISHING

Once the material has set, it can be finished and polished with the usual method. We recommend the use of **POLI-R** for polishing and **POLI-B** to obtain a high shine.

SALES FORMAT

Kit Standard

9030-100	3 x 3 g syringe
	+ 1 x 50 ml Bonder
	+ 12 app. tips

Accessories:

9030-100 UV-LIGHT LAMP



protechno

ADVANCED PRODUCTS FOR DENTAL LABS



Tel. +34 972 526 169

Polígono Empordà Internacional
17469 . VILAMALLA (Girona) . Spain

protechno@protechno.com
www.protechno.com