



CONTACT-SPRAY

Descripción

Spray de oclusión para la visualización de los puntos de contacto.

Utilización

Para la localización de los puntos de contacto en los trabajos de prótesis.

Ventajas

- Se obtiene una película de grueso ultrafino y uniforme
- Pigmentos extra-finos para dosificaciones exactas.
- Fácil de eliminar
- Sin CFC

Modo de empleo

Agitar antes de usar.

Introducir el pulsador en el cabezal de la válvula.

Aproximar el extremo de la cánula a aprox. 3 cm del objeto.

Apretar el pulsador ligeramente y después pulverizar rotando el objeto

Se puede eliminar fácilmente con un pincel, vapor de agua o agua corriente.

Precauciones

Recipiente spray a presión

Proteger del sol y de temperaturas superiores a 50 °C

No incinerar ni agujerear una vez acabado el producto.

No pulverizar sobre llamas ni materiales incandescentes.

No fumar

Guardar fuera del alcance de los niños

Almacenaje y caducidad

La estabilidad del producto es de 3 años, respetándose las condiciones de almacenamiento.

La fecha de caducidad se entiende con el producto en su envase original y previo a la apertura del mismo

Almacenaje: Conservar el producto bien cerrado en su embalaje original y en un lugar fresco y seco

Presentación

3245-750	75 ml. Spray verde
3245-751	75 ml. Spray azul
3245-752	75 ml. Spray rojo
3245-753	75 ml. Spray verde fluor

Información adicional

Las informaciones contenidas en esta hoja de instrucciones están basadas en el estado actual de nuestros conocimientos y se dan solamente a título informativo sin que ello implique una garantía por parte de Protechno.



protechno

Polígono Empordà Internacional
17469 VILAMALLA, GIRONA, SPAIN
Tel.: (34) 972 52 61 69
protechno@protechno.com
www.protechno.com



CONTACT-SPRAY

Description

Spray for marking contact points visible.

Use

To visualize pre contact points and for adjustment work on crowns and bridges

Advantages

- A film of ultra-thin and uniform thickness is obtained
- Extra-fine pigments for exact dosages.
- Easy to remove
- CFC-free

Application

Shake well before use.

Spray at a distance of app. 3 cm.

Soft, intermittent short bursts while turning the sprayed object.

Can be removed easily with a brush, water vapor or running water.

Precautions

Container with pressure

Protect from sunshine and temperatures above 50 °C

Do not incinerate or pierce once the product is finished.

Do not spray on flames or materials incandescent.

No Smoking

Store out of reach of children

Storage and stability

The stability of the product is 3 years respecting the storage conditions.

Note: The expiry date is considered with the product in his original packing and before been opened.

Storage: Keep the product closed in its original package and in a dry cool place.

Packing

3245-750	75 ml. Spray - Green
3245-751	75 ml. Spray - Blue
3245-752	75 ml. Spray - Red
3245-753	75 ml. Spray - Green fluor

Additional information

The information given in this sheet is based on our level of knowledge at the moment at the date of printing and is not intended to be a warranty for the product.



protechno

Polígono Empordà Internacional
17469 VILAMALLA, GIRONA , SPAIN
Tel.: (34) 972 52 61 69
protechno@protechno.com
www.protechno.com