MICRO-LAC

Description

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Utilisation

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Dye varnish for painting stumps

Uses

Varnish for painting stumps, serving as separator for the cement, at the same time sealing the stone surface.

Advantages:

- Wide range of thickness from 5 to 13 µm
- Bright colours that contrast to waxes and porcelains.
- Hard and smooth surfaces after setting.
- Very good adhesion to stone models: it's impossible to remove with steam.

Use instructions

Shake the bottle vigorously before use until the uniformity of the product

Remove the excess of liquid from the brush and apply directly on the stone stump.

Let dry completely before applying a new layer

The setting time can be modified by the room temperature and the amount of applied product (approx. 15 sec)

If it's necessary, some drops of thinner can be added to the spacer to increase the viscosity.

Safety cautions

High flammable product. Work in well ventilated places. Keep away from naked flames, hot items and sparks
Do not eat, drink or smoke while using.

Avoid contact with skin or eyes and the inhalation of gases. In the event of ingestion, consult a doctor
For further information, see the safety data sheet of the product

Storage

The stability of the product is 2 years respecting the storage conditions. (Note: The expiry date is considered with the product in his original packing and before been opened).

Storage: Keep the product tightly closed in its original package and in a dry cool place.

Packing

3221-300	Blue 5 µm	Bottle 30 ml
3221-301	Red 10 µm	Bottle 30 ml
3221-302	Silver 13 µm	Bottle 30 ml
3222-300	Thinner	Bottle 30 ml

Additional information

The information given in this sheet is based on our level of knowledge at the moment at the date of printing and is not intended to be a warranty for the product.

Polígono Empordà Internacional 17469 VILAMALLA, GIRONA, SPAIN
 Tel.: (34) 972 52 61 69
 protechno@protechno.com www.protechno.com

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The setting time can be modified by the room temperature and the amount of applied product (approx. 15 sec) If it's necessary, some drops of thinner can be added to the spacer to increase the viscosity.

Safety cautions

High flammable product. Work in well ventilated places. Keep away from naked flames, hot items and sparks Do not eat, drink or smoke while using.

Avoid contact with skin or eyes and the inhalation of gases. In the event of ingestion, consult a doctor For further information, see the safety data sheet of the product

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