YESOS · STONES · PLÂTRES · GESSI

DIAMANT

| | | |
|---|----|-------|
| HOJA DE INSTRUCCIONES | ES | 3 |
| WORKING INSTRUCTIONS | EN | 4 |
| NOTICE D'EMPLOI | FR | 5 |
| ISTRUZIONI PER L'USO | IT | 6 |
| INFORMACIÓN ADICIONAL ADDITIONAL INFORMATION INFORMATIONS COMPLÉMENTAIRES INFORMAZIONI SUPPLEMENTARI | | 2 & 7 |

Consulte nuestra web
Visit our website

www.protechno.com

CARACTERÍSTICAS FÍSICAS · PHYSICAL CHARACTERISTICS
 CARACTÉRISTIQUES PHYSIQUES · CARATTERISTICHE FISICHE

100 g / 20 ml

Proporción polvo/agua desmineralizada
 Powder/ demineralised water ratio
 Dosage poudre/eau déminéralisée
 Proporzioni polvere/acqua demineralizzata

20°C - 23°C

Temperatura de trabajo
 Working temperature
 Température de travail
 Temperatura di lavorazione

60''

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DIAMANT

DESCRIPTION

Extra hard synthetic die-stone. Type IV, high quality ISO 6873.

USES Ideal for all types of fixed prostheses, stumps and master models.

ADVANTAGES This product satisfies the most high requirements of the professionals:

- Great dimensional stability.
- Hard and even surfaces.
- Great resistance to scratches and abrasion.
- Good definition of details.
- High resistance to breaking.
- Thixotropic gypsum with good consistence for an easy handling and casting.

INSTRUCTIONS OF USE

Add the powder to the demineralised water following the recommended ratio. The initial consistency of the slurry must be thick. In this moment don't add more water above the recommended. While the mixing time is complete, the product acquires the suitable fluidity and stability. When this mixing time has finished, the mix must have a soft and high thixotropic consistency, flowing easily over the model with a low vibration. After 4 minutes, stop the vibration to guarantee a good crystallisation of the product.

CAUTIONS

1.- USE CAUTIONS

Use always-clean instruments. Do not mix the product in a container where investments have been mixed. Use always demineralised water to do the mix. The water from the public mains can affect the correct setting process of the gypsum, making it faster or slower.

2.- SAFETY CAUTIONS

Do not inhale the powder. It's recommended the use of a protective mask. Keep away from children. Handle the product in a well-ventilated place. The handling of this product has to be done only by qualified people. In the event of ingestion, consult a doctor. For further information, see the safety data sheet for the product.

EXPIRY AND STORAGE

EXPIRY

The stability of the product is 18-36 months depending on the package and respecting the storage conditions. (Note: The expiry date is considered with the product in his original packing and before been opened).

STORAGE

Keep the packages tightly closed in a dry place away from moisture. After each use the package must be closed to prevent the moisture absorption.

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EN

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet of DIAMANT. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review.

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