YESOS • STONES • GESSI • PLÂTRES

KIMBERLIT

HOJA DE INSTRUCCIONES	ES	3
WORKING INSTRUCTIONS	EN	4
ISTRUZIONI PER L'USO	IT	5
NOTICE D'EMPLOI	FR	6
INFORMACIÓN ADICIONAL ADDITIONAL INFORMATION INFORMAZIONI SUPPLEMENTARI INFORMATIONS COMPLÉMENTAIRES		2 & 7

Consulte nuestra web
Visit our website

CARACTERÍSTICAS FÍSICAS • PHYSICAL CHARACTERISTICS CARATTERISTICHE FISICHE • CARACTÉRISTIQUES PHYSIQUES

100 g / 20 ml

Proporción polvo/agua desmineralizada
Powder/ demineralised water ratio
Proporzioni polvere/acqua demineralizzata
Dosage poudre/eau déminéralisée

20° - 23° C

Temperatura de trabajo
Working temperature
Temperatura di lavorazione
Température de travail

60''

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DESCRIPTION

Extra hard natural die-stone. Type IV, high quality ISO 6873.

USES

Ideal for all types of fixed prostheses, stumps and master models.

ADVANTAGES

This product satisfies the highest requirements of the professionals:

- Great dimensional stability
- Hard and even surfaces
- Great resistance to scratches and abrasion
- Good definition of details
- High resistance to breaking
- Thyrotrophic gypsum with good consistence for an easy handling and casting

INSTRUCTIONS OF USE

Add the powder to the demineralised water following the recommended ratio. The initial consistency of the slurry must be thick. In this moment don't add more water above the recommended.

While the mixing time is complete, the product acquires the suitable fluidity and stability. When this mixing time has finished, the mix must have a soft and high thyrotrophic consistency, flowing easily over the model with a low vibration.

After 4 minutes, stop the vibration to guarantee a good crystallisation of the product.

CAUTIONS

1.- USE CAUTIONS

Use always-clean instruments.
Do not mix the product in a container where investments have been mixed.
Use always demineralised water to do the mix. The water from the public mains can affect the correct setting process of the gypsum, making it faster or slower.

2.- SAFETY CAUTIONS

Do not inhale the powder. It's recommended the use of a protective mask.
Keep away from children.
Handle the product in a well-ventilated place.
The handling of this product has to be done only by qualified people.
In the event of ingestion, consult a doctor.
For further information, see the safety data sheet for the product.

EXPIRY AND STORAGE

EXPIRY

The stability of the product is 18-36 months depending on the package and respecting the storage conditions. (Note: The expiry date is considered with the product in his original packing and before been opened).

STORAGE

Keep the packages tightly closed in a dry place away from moisture.
After each use the package must be closed to prevent the moisture absorption.

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EN

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet of KIMBERLIT. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review.

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