YESOS · STONES · PLÂTRES · GESSI

RUBINIT

HOJA DE INSTRUCCIONES	ES	3
WORKING INSTRUCTIONS	EN	4
NOTICE D'EMPLOI	FR	5
ISTRUZIONI PER L'USO	IT	6
INFORMACIÓN ADICIONAL ADDITIONAL INFORMATION INFORMATIONS COMPLÉMENTAIRES INFORMAZIONI SUPPLEMENTARI		2 & 7

Consulte nuestra web
Visit our website

www.protechno.com

CARACTERÍSTICAS FÍSICAS · PHYSICAL CHARACTERISTICS
CARACTÉRISTIQUES PHYSIQUES · CARATTERISTICHE FISICHE

100 g / 22 -24 ml

Proporción polvo/agua desmineralizada
 Powder/ demineralised water ratio
 Dosage poudre/eau déminéralisée
 Proporzioni polvere/acqua demineralizzata

20°C - 23°C

Temperatura de trabajo
 Working temperature
 Température de travail
 Temperatura di lavorazione

60''

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RUBINIT

DESCRIPTION

Extra-hard natural stone. Type IV, high quality ISO 6873.

USES

For the fabrication of models, slumps and bases in fixed and removal prostheses.

ADVANTAGE S

- Great dimensional stability.
- Very smooth and hard surfaces.
- High resistance to scratches and abrasion.
- Excellent detail definition.
- Compatible with all printing materials.
- Thixotropic plaster with optimal consistency for casting and handling.

INSTRUCTIONS OF USE

Add the powder to the demineralised water following the recommended ratio. The initial consistency of the slurry must be thick. In this moment don't add more water above the recommended. While the mixing time is complete, the product acquires the suitable fluidity and stability. When this mixing time has finished, the mix must have a soft and high thixotropic consistency, flowing easily over the model with a low vibration. After 4 minutes, stop the vibration to guarantee a good crystallisation of the product.

CAUTIONS

1.- USE CAUTIONS

Use always-clean instruments. Do not mix the product in a container where investments have been mixed. Use always demineralised water to do the mix. The water from the public mains can affect the correct setting process of the gypsum, making it faster or slower.

2.- SAFETY CAUTIONS

Do not inhale the powder. It's recommended the use of a protective mask. Keep away from children. Handle the product in a well-ventilated place. The handling of this product has to be done only by qualified people. In the event of ingestion, consult a doctor. For further information, see the safety data sheet for the product.

EXPIRY AND STORAGE

EXPIRY

The stability of the product is 18-36 months depending on the package and respecting the storage conditions. (Note: The expiry date is considered with the product in his original packing and before been opened).

STORAGE

Keep the packages tightly closed in a dry place away from moisture. After each use the package must be closed to prevent the moisture absorption.

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EN

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet of RUBINIT. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review.

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