TECHNOPAINT

HOJA DE INSTRUCCIONES	ES	2
WORKING INSTRUCTIONS	EN	3
NOTICE D'EMPLOI	FR	4
ISTRUZIONI PER L'USO	IT	5
INFORMACIÓN ADICIONAL ADDITIONAL INFORMATION INFORMATIONS COMPLÉMENTAIRES INFORMAZIONI SUPPLEMENTARI		6

Consulte nuestra web
Visit our website

TECHNOPAINT

DESCRIPCIÓN

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EN

DESCRIPTION

Fine investment material for partial denture technique.

USES

Fine investment for painting the wax or plastic patterns in the frame work technique. Application before filling the cylinder with the usual investment.

ADVANTAGES

- Avoids the defects on the castings
- It assures a fine casting surface
- Quick setting (1 minute)

INSTRUCTIONS OF USE

Due to the composition of the product, during long storage periods, the product can settle. It's recommended to shake it well until a uniform product is obtained.

Apply the product with a brush onto the wax patterns.

Let dry completely approximately for 1 minute.

Fill the cylinder with the usual phosphate or polysilicate investment.

Close the bottle after each use.

If the product is too thick, some distilled water can be added until the correct viscosity is obtained.

STORAGE AND EXPIRY

The stability of the product is 18 months respecting the storage conditions.

Note: The expiry date is considered with the product in his original packing and before been opened.

Storage: Keep the product tightly closed in its original package and in a dry cool place.

SAFETY INSTRUCTIONS

Work in well ventilated places.

Do not eat, drink or smoke while using.

Avoid contact with skin or eyes.

In the event of ingestion, consult a doctor.

For further information, see the safety data sheet of the product.

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EN

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet of TECHNOPAINT. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review.

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