# ECO-CRYL M

HOJA DE INSTRUCCIONES	ES	2
WORKING INSTRUCTIONS	EN	4
ISTRUZIONI PER L'USO	IT	6
NOTICE D'EMPLOI	FR	8
INFORMACIÓN ADICIONAL ADDITIONAL INFORMATION INFORMAZIONI SUPPLEMENTARI INFORMATIONS COMPLÉMENTAIRES		10

CE  
1639

# ECO-CRYL M

## DESCRIPCIÓN

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# ECO-CRYL M

## DESCRIPTION

ECO-CRYL M is a heat curing acrylic resin for total and partial denture bases.

Type 5, ISO 20795-1:2013.

## PROPERTIES

Various polymerisation possibilities: traditional method with boiling water or direct into microwave.  
Plastic and malleable consistency, cadmium free and permanent colour.  
Resistant to alcohol, fruit drinks and various brands of denture cleaners.  
Can be used with any kind of acrylic or porcelain teeth.

## INSTRUCTIONS OF USE

### WORKING TIME

#### Dough Time

10-20 minutes, at 22 °C and with the following proportion:

- For 1 prosthesis: 20 g powder / 10 ml liquid
- For 2 prostheses: 40 g powder / 20 ml liquid

Use the dosers supplied with the product.

#### Working Time

As soon as the material has doughed sufficiently and not longer sticks to the fingers, it can be worked for 15 minutes, with a room temperature of 22°C.

### APPLICATION METHOD

#### MOULD PREPARATION

Prepare the plaster mould in a denture flask having spring closure, remove the wax by scalding it in hot water. To cure in microwaves, use the appropriate muffle as the PROTECHNO's MUFLA MICROONDAS.

Paint the mould with a separating varnish such as TECHNOSIL or TECHNOSIL-SUPER, taking care not to paint the teeth. Leave to dry.

For the upper part of the flask, we recommend MUFLA-GIPS. It has a fluid consistency and vibration is not necessary. This plaster does not adhere to dentures neither to acrylic teeth.

To ensure an appropriate bond with the denture base, routhen the teeth with a bur and wet with monomer.

#### PREPARING THE DOUGH

Measure out the powder and liquid in the recommended ratio.

Mix with a spatula for 60 seconds, and ensure that air is not trapped in the mixture.

Cover the receptacle and leave to rest for 10-20 minutes until the mixture gets doughed. In this point the mixture is malleable and does not stick to ones fingers.

The mixture remains plastic for a maximum of 30 minutes.

(with a covered receptacle and at a temperature of 22 °C)

## APPLICATION METHOD

### PACKING AND CLOSING THE MOULD

Place the dough in the warm mould (about 40 °C) until there is an excess, and then cover it with a cellophane sheet. Close the mould slowly and open to remove the excess dough. Close again slowly after removing the cellophane sheet and under a press, leaving to rest for some 10 minutes.

### POLYMERISATION

#### a) **Standard cure:**

Place the flask in warm water (30-40 °C). Heat up to 100 °C and leave to boil for 20 minutes.

Ensure that there are at least 3 liters of water in the receptacle. Where several flasks are polymerised at the same time, there should be at least 2 liters of water per flask.

After the curing process has been completed, leave to cool the flask for at least 30 minutes exposed to the air. It may then be cooled with in cold water.

#### b) **Microwave cure:** Place the flask in the microwave oven and heat for 3 minutes at 700 W.

After the curing process has been completed the flask can be put directly into cold water for 45 minutes. The MUFLA MICROONDAS (microwave muffle) is made of plastic material which is not temperature conductive and that ensures a progressive cool down without resin contractions.

## FINISHING AND POLISHING

ECO-CRYL M can be finished and polished using usual methods. We recommend the use of the polishing pastes POLI-R and POLI-B.

## TECHNICAL INFORMATION

Residual monomer content for ECO-CRYL M when cured with the above microwave cure method is < 1.5 % (test 8.8 ISO 20795-1:2013).

## STORAGE AND EXPIRY

ECO-CRYL M powder and liquid retain all those qualities over a period of at least 2 years.

- Batch number and expiry date are marked on each package
- Store in a dry place at a temperature not exceeding 25 °C
- Keep out of sunlight

## SAFETY INSTRUCTIONS

ECO-CRYL M liquid contains flammable solvents and should be handled in well-ventilated non-smoking areas, away from heat and naked flames.

DO NOT INHALE THE VAPOURS.

Contact with the skin and eyes should be avoided and suitable protective clothing should be worn, i.e. overalls, gloves and goggles. In the event of contact wash immediately with copious amounts of water. A spillage of liquid should be cleaned up immediately with sand or earth.

In the event of fire use sand, earth or a suitable extinguisher.

Store in a cool place from direct sunlight, naked flames and sparks.

Flashpoint: 10° C.

### Contraindication and side effects

Do not use if the patient is known to be allergic to any of the ingredients in ECO-CRYL M.

In individual cases, local allergic reactions to methacrylate materials have been reported.

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EN

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet for products ECO-CRYL M powder and liquid. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review.

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