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ECO-CRYL COLD

HOJA DE INSTRUCCIONES

ES

2

WORKING INSTRUCTIONS

EN

4

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IT

6

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FR

8

INFORMACIÓN ADICIONAL

10

ADDITIONAL INFORMATION

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INFORMATIONS COMPLÉMENTAIRES

CE
1639

MD

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ECO-CRYL COLD

DESCRIPTION

Intended purpose: Denture repair and relining material. Self curing

Classification: Type 2, Class 1, ISO 20795-1:2013

Indications: Partial or complete edentulism and dental trauma

Intended patient population: Patients with partial or complete edentulism and dental trauma

Medical conditions: Edentulous or partially toothless patients with no contraindication

Intended user: Dental technician

Intended environment: Dental prosthetic laboratory

Clinical benefits: Replacement of missing teeth

COMPOSITION

Powder / liquid cold cure resin.

Powder: 98% methyl methacrylate-copolymer

Liquid: 94% methyl methacrylate-monomer

INSTRUCTIONS OF USE

WORKING TIME

Measure out the powder and liquid in the following ratio:

21 g powder : 18 ml liquid

Pour the liquid in a mixing receptacle, add the powder and mix it without producing air bubbles.

Plastic phase

2-5 minutes at 20-21°C.

RECOMMENDED PRODUCT COMBINATION

Acrylic teeth and general-purpose equipment habitual in the prosthetic laboratory.

This device is NOT intended to be used with porcelain teeth.

APPLICATION METHOD

MOULD PREPARATION

To assure a good insulate of the resin, put the gypsum mould in hot water and, once dry, paint the surface with a spacing varnish as TECHNO SIL. Set up the repair on a plaster model, and clean up the edges of the fracture.

PREPARING THE DOUGH

- Measure out the powder and liquid in the recommended proportion
- Free measurement: Pour the liquid in a mixing receptacle, add the powder until there is a fine layer of unwetted powder on the surface, wait 30 seconds and shake the receptacle to wet all of the powder.

APPLICATION METHOD

APPLICATION

Transfer the mixture of the repair using a thin bladed spatula building to the thickness required. After 2-5 minutes it will reach the plastic phase, when the mould would not be handled more.

ECO-CRYL COLD can be used also under pressure; this polymerisation allows free-pore repairing.

Once hardened it can be polished.

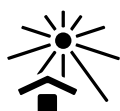
FINISHING AND POLISHING

ECO-CRYL COLD can be finished and polished using usual methods. We recommend the use of POLI-R pastes for grinding and polishing and POLI-B for the final shine.

TECHNICAL INFORMATION

Residual monomer content for ECO-CRYL COLD when cured with standard method is < 4.5 % (test 8.8 ISO 20795-1:2013).

STORAGE AND EXPIRY

ECO-CRYL COLD powder and liquid retain all those qualities over a period of at least 2 years when stored in their original packaging, in a moisture-free environment, at a temperature not exceeding 25°C, and out of reach of sunlight.

- Batch number and expiry date are marked on each package
- Store in a dry place at a temperature not exceeding 25 °C
- The device must remain in its sealed container away from sunlight

SAFETY INSTRUCTIONS

Contient: methyl methacrylate.

Flash point: 10° C

- Highly flammable liquid and vapour.
- Causes skin irritation.
- May cause an allergic skin reaction.
- May cause respiratory irritation.
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources.
- No smoking.
- Avoid breathing vapours.
- Wear protective clothing, eye protection, face protection.
- IF ON SKIN: Wash with plenty of soap and water.
- Dispose of contents and container to an approved waste disposal plant.

Contraindication and side effects

Do not use if the patient is known to be allergic to any of the ingredients in ECO-CRYL COLD
In individual cases, local allergic reactions to methacrylate materials have been reported.

Safe disposal of the device

For monomer and polymer, disposal should be according to regulation as stated in their SDS. For final polymerised product, no special measures are required. They are not considered hazardous waste, so they can be placed in regular waste.

ADDITIONAL INFORMATION

This device is single use. There is no risk of reuse as the material, once polymerised, cannot become powder and liquid again.

Information for dental professional to patients

- The product, before and after polymerization (prothesis), should not be disinfected using microwave, as it can affect the resin structure.
- Clean the product with non-aggressive and non-abrasive dental cleaning products
- Patients using the device should have periodic check-ups by a dental professional

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EN

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet for products ECO-CRYL COLD powder and liquid. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review. (<https://brochures.protechno.com/Docs//HI/06/HI-T-ECOCRYL-COLD.pdf>)

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For use only by a dental technician in the recommended indications.

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Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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