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ECO-CRYL HOT

HOJA DE INSTRUCCIONES

ES

2

WORKING INSTRUCTIONS

EN

4

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IT

6

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FR

8

INFORMACIÓN ADICIONAL

10

ADDITIONAL INFORMATION

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INFORMATIONS COMPLÉMENTAIRES

CE
1639

MD

ADVANCED PRODUCTS FOR DENTAL LABS

HI-T ECO-CRYL HOT PROTECHNO V.1.6 / R.21/03/2024

DESCRIPCIÓN

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DESCRIPTION

- Intended purpose:** Denture material for total and partial prostheses. Hot curing
- Classification:** Type 1 Class 1, ISO 20795-1:2013
- Indications:** Partial or complete edentulism and dental trauma
- Intended patient population:** Patients with partial or complete edentulism and dental trauma
- Medical conditions:** Edentulous or partially toothless patients with no contraindication
- Intended user:** Dental technician
- Intended environment:** Dental prosthetic laboratory
- Clinical benefits:** Replacement of missing teeth

COMPOSITION

Acrylic resin in liquid and powder form.

Powder:	98% methyl methacrylate-copolymer
Liquid:	96% methyl methacrylate-monomer

INSTRUCTIONS OF USE

WORKING TIME

. For 1 prosthesis: 24 g powder / 13-14 ml liquid

. For 2 prostheses: 48 g powder / 26- 28 ml liquid

Use the dosers supplied with the product.

Dough/packing time

Initial packing time: from minute 15-20 (20°C)

Final packing time: up to minute 30 (20° C)

RECOMMENDED PRODUCT COMBINATION

Acrylic teeth and general-purpose equipment habitual in the prosthetic laboratory.
This device is NOT intended to be used with porcelain teeth.

APPLICATION METHOD

MOULD PREPARATION

Prepare the plaster mould in a denture flask having spring closure, remove the wax by scalding it in hot water. Paint the warm mould with a separating varnish such as TECHNOSIL, taking care not to paint the teeth. Leave to dry and apply a second layer if necessary.

To ensure an appropriate bond with the denture base, roughen the teeth with a bur and wet with monomer.

PREPARING THE DOUGH

- Measure out the powder and liquid in the recommended proportion
- Free measurement: Pour the liquid in a mixing receptacle, add the powder until there is a fine layer of unwetted powder on the surface, wait 30 seconds and shake the receptacle to wet all of the powder.

Mix with a spatula for 60 seconds, and ensure that no air is trapped in the mixture. The relatively fluid consistency of the mixture is misleading - there is NO need to add more powder.

Cover the receptacle and leave to rest for 15-20 minutes until the mixture gets doughed. In this point the mixture is malleable and does not stick to one's fingers.

The mixture remains plastic during the dough/packing time, no more than 30-35 minutes from the beginning of the mixing.

PACKING AND CLOSING THE MOULD

Place the dough in the warm mould (about 40 °C) until there is an excess, and then cover it with a cellophane sheet. Close the mould slowly and open to remove the excess dough. Close again slowly after removing the cellophane sheet and press (200 kg/cm²) leave to rest for some 10 minutes and fix with a clamp

APPLICATION METHOD

POLYMERISATION

- Standard cure:** Place the flask in cold water, heat up to 80 °C and maintain for 2 hour and then allow to boil at 100°C for a further 30 minutes.
- Fast cure:** Place the flask in warm water (30-40 °C). Heat up to 100 °C and leave to boil for 45 minutes.
- Long cure:** Place the flask in a curing device, fill with cold water and leave for 10 hours at 80 °C

After the curing process has been completed with any of the above systems, let the flask cool for at least 30 minutes exposed to the air. It may then be cooled with in cold water.

Ensure that there are at least 3 litres of water in the receptacle. Where several flasks are polymerised at the same time, there should be at least 2 litres of water per flask.

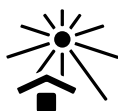
FINISHING AND POLISHING

ECO-CRYL HOT can be finished and polished using usual methods. We recommend the use of POLI-R pastes for grinding and polishing and POLI-B for the final shine.

TECHNICAL INFORMATION

Residual monomer content for ECO-CRYL HOT when cured with standard method is < 0.5 % (test 8.8 ISO 20795-1:2013).

STORAGE AND EXPIRY

ECO-CRYL HOT powder and liquid retain all those qualities over a period of at least 3 years when stored in their original packaging, in a moisture-free environment, at a temperature not exceeding 25°C, and out of reach of sunlight.

- Batch number and expiry date are marked on each package
- Store in a dry place at a temperature not exceeding 25 °C
- The device must remain in its sealed container away from sunlight

SAFETY INSTRUCTIONS

Contient: methyl methacrylate.

Flash point: 10° C

- Highly flammable liquid and vapour.
- Causes skin irritation.
- May cause an allergic skin reaction.
- May cause respiratory irritation.
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources.
- No smoking.
- Avoid breathing vapours.
- Wear protective clothing, eye protection, face protection.
- IF ON SKIN: Wash with plenty of soap and water.
- Dispose of contents and container to an approved waste disposal plant.

Contraindication and side effects

Do not use if the patient is known to be allergic to any of the ingredients in ECO-CRYL HOT. In individual cases, local allergic reactions to methacrylate materials have been reported.

Safe disposal of the device

For monomer and polymer, disposal should be according to regulation as stated in their SDS. For final polymerised product, no special measures are required. They are not considered hazardous waste, so they can be placed in regular waste.

ADDITIONAL INFORMATION

This device is single use. There is no risk of reuse as the material, once polymerised, cannot become powder and liquid again.

Information for dental professional to patients

- The product, before and after polymerization (prosthesis), should not be disinfected using microwave, as it can affect the resin structure.
- Clean the product with non-aggressive and non-abrasive dental cleaning products
- Patients using the device should have periodic check-ups by a dental professional

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EN

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet for products ECO-CRYL HOT powder and liquid. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review. (<https://brochures.protechno.com/Docs//HI/06/HI-T-ECOCRYL-HOT.pdf>)

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For use only by a dental technician in the recommended indications.

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Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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