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ECO-CRYL PRO

HOJA DE INSTRUCCIONES	ES	2
WORKING INSTRUCTIONS	EN	4
ISTRUZIONI PER L'USO	IT	6
NOTICE D'EMPLOI	FR	8
INFORMACIÓN ADICIONAL ADDITIONAL INFORMATION INFORMAZIONI SUPPLEMENTARI INFORMATIONS COMPLÉMENTAIRES		10

CE
1639

MD

ECO-CRYL PRO

DESCRIPCIÓN

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ECO-CRYL PRO

DESCRIPTION

- Intended purpose:** Denture material for total and partial prostheses by pouring technique. Self-curing
- Classification:** Type 2, Class 2, ISO 20795-1:2013
- Indications:** Partial or complete edentulisme and dental trauma
- Intended patient population:** Patients with partial or complete edentulisme and dental trauma
- Medical conditions:** Edentulous or partially toothless patients with no contraindication
- Intended user:** Dental technician
- Intended environment:** Dental prosthetic laboratory
- Clinical benefits:** Replacement of missing teeth

INSTRUCTIONS OF USE

TECHNICAL SPECIFICATIONS

Mixing proportion:

For 1 prosthesis: 25 g powder / 19 ml liquid

Use the dosers supplied with the product.

Mixing Time:	30 sec
Plastic phase:	2-5 min
Curing under pressure:	35 min. (45° C / 2 bar)

Times depend on the room temperature. A higher temperature will reduce the working time, a lower temperature will increase the pouring time.
These values are given with a room temperature of 20-22°C.

RECOMMENDED PRODUCT COMBINATION

Acrylic teeth and general-purpose equipment habitual in the prosthetic laboratory.
This device is NOT intended to be used with porcelain teeth.

APPLICATION METHOD

ECO-CRYL PRO is specially designed for the production of dentures using the gelatin or silicone pouring technique. We recommend the use of the POURING FLASK and TECHNOJEL-PRO gelatin.

MOULD PREPARATION

Prepare the wax model on the plaster mould. Fix it to the muffle baseplate with wax or the magnet. Close the flask and fix it with the yellow rubber band. Place also the three-plug piece.
Fill in the MUFLA DE VERTIDO through the central hole with the TECHNOJEL-PRO and let it cool down about 1 hour.
Remove the model and clean the wax from the teeth with hot water and soap. Use the cleaning box supplied (is the base of the flask) to place the dental pieces in order during the wax-up. Fix the cleaning box with the short rubber band.

Once the teeth are clean, apply the needed retentions and set them back in their place.
Pierce two pouring channels with the cutting tubes

Soak the model in warm water until no air bubbles come out. Dry the surface and paint it with a separating varnish such as TECHNOSIL, taking care not to paint the teeth. Place the model back inside the flask and close with the rubber band.

APPLICATION METHOD

PREPARING THE RESIN

Measure out the powder and liquid in the recommended proportion.
Mix with a spatula for 30 seconds, and ensure that air is not trapped in the mixture.

ECO-CRYL PRO can be poured for 2-5 minutes

Pour the resin through one of the pouring channel until the resin overflows through the other.

POLYMERISATION

Place the flask in a pressure polymerizer with the holes upwards. Fill in with warm water (45°C) up to the edge of the flask.
Cure for 35 minutes with 2-3 bar of pressure

FINISHING AND POLISHING

Once the polymerization has finished, deflask the model and remove the spill-overs.

ECO-CRYL PRO can be finished and polished using usual methods.

We recommend the use of POLI-R pastes for grinding and polishing and POLI-B for the final shine.

TECHNICAL INFORMATION

Residual monomer content for ECO-CRYL PRO when cured with standard method is < 4.5 % (test 8.8 ISO 20795-1:2013).

STORAGE AND EXPIRY

ECO-CRYL PRO powder and liquid retain all those qualities over a period of at least 2 years when stored in their original packaging, in a moisture-free environment, at a temperature not exceeding 25°C, and out of reach of sunlight.

- Batch number and expiry date are marked on each package
- Store in a dry place at a temperature not exceeding 25 °C
- The device must remain in its sealed container away from sunlight

SAFETY INSTRUCTIONS

Content: methyl methacrylate.

Flash point: 10° C

- Highly flammable liquid and vapour.
- Causes skin irritation.
- May cause an allergic skin reaction.
- May cause respiratory irritation.
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources.
- No smoking.
- Avoid breathing vapours.
- Wear protective clothing, eye protection, face protection.
- IF ON SKIN: Wash with plenty of soap and water.
- Dispose of contents and container to an approved waste disposal plant.

Contraindication and side effects

Do not use if the patient is known to be allergic to any of the ingredients in ECO-CRYL PRO.
In individual cases, local allergic reactions to methacrylate materials have been reported.

Safe disposal of the device

For monomer and polymer, disposal should be according to regulation as stated in their SDS. For final polymerised product, no special measures are required. They are not considered hazardous waste, so they can be placed in regular waste.

ADDITIONAL INFORMATION

This device is single use. There is no risk of reuse as the material, once polymerised, cannot become powder and liquid again.

Information for dental professional to patients

- The product, before and after polymerization (prothesis), should not be disinfected using microwave, as it can affect the resin structure.
- Clean the product with non-aggressive and non-abrasive dental cleaning products
- Patients using the device should have periodic check-ups by a dental professional

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EN

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet for products ECO-CRYL PRO powder and liquid. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review. (<https://brochures.protechno.com/Docs//HI/06/HI-T-ECOCRYL-PRO.pdf>)

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For use only by a dental technician in the recommended indications.

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Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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