ECO-TRAY LUZ

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WORKING INSTRUCTIONS EN 4

INFORMACIÓN ADICIONAL 6
ADDITIONAL INFORMATION

CE

MD

Consulte nuestra web
Visit our website

DESCRIPCIÓN

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ECO-TRAY LUZ

DESCRIPTION

The product is a light-curing tray material in the form of prefabricated, upper baseplates to make rigid, custom impression trays and bite-blocks, as well as for other indications. ECO-TRAY LUZ can be cured in all ultraviolet and Halogen light-curing units. The baseplates are supplied in a reusable light-proof box that can be used for transporting or storing models, etc.

INDICATIONS

- Custom impression tray
- Bite wafer for try-in framework
- Key for supporting pin registration
- Bite-block
- Splinting outer coping or telescoping crowns
- Baseplate for setting up dentures

USE INDICATIONS

PREPARATION

- Mark the outline of the future tray on the model
- Mask the undercuts on the model
- To create a spacer for the impression material, coat the entire model by means of a sheet of wax
- If the plaster is very dry, sprinkle the model slightly with water or apply an alginate insulation

APPLICATION

- Take the baseplate from the box and close it
- Once the model has been coated, adapt the baseplates and cut to shape
- Use the remaining material to form a handle and attach it to the tray
- Place the model and tray in a light-curing unit and cure without vacuum
- Remove the tray from the model and polymerize at the inside
- Use alcohol to remove the smear, tacky layer

WORKING TIME

Working at daylight: **20 minutes**

Polymerisation in an ultraviolet light-curing unit

1. the top of the tray, on the model: **3-5 minutes**
2. the inside of the tray, without model: **3-5 minutes**

TRIMMING

- Always use a dust extractor when trimming the tray!
- Trim the custom tray with tungsten carbide burs and smooth it with sandpaper.
- Pumice slurry can be used for polishing. We recommend our product POLI-R.

HYGIENE

Dip the impressions taken with this material custom trays in a disinfectant solution for the time specific for the disinfectant.

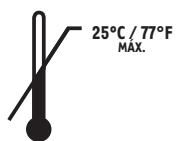
INCOMPATIBILITY

This material may cause sensitization in sensitive persons, if such reactions are experienced, discontinue the use of this material.

TECHNICAL DATA

Flexural strength: **120N/mm²**

STORAGE

- Do not store this material above 25°C/77°F
- Protect against light sources

SHELF-LIFE

The product has a lifespan of 3 years from the manufacturing date, provided it is kept in its original unopened packaging and stored according to the storage instructions.

Do not use ECO-TRAY LUZ after the expiry date.

WASTE DISPOSAL

Avoid release to the environment, dispose as special waste. Comply with local Regulations in force.

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EN

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet for product ECO-TRAY LUZ. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review.

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For use only by a dental professional in the recommended indications.

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Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

🏢 Famadent S.L.U.
Polígono Empordà Internacional
17469 Vilamalla (Girona) • SPAIN
Tel.: +34 972 52 61 69
protechno@protechno.com

www.protechno.com