ECO-CRYL HOT

HOJA DE INSTRUCCIONES	ES	2
WORKING INSTRUCTIONS	EN	4
ISTRUZIONI PER L'USO	IT	6
NOTICE D'EMPLOI	FR	8
INFORMACIÓN ADICIONAL ADDITIONAL INFORMATION INFORMAZIONI SUPPLEMENTARI INFORMATIONS COMPLÉMENTAIRES		10

CE
1639

ECO-CRYL HOT

DESCRIPCIÓN

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ECO-CRYL HOT

DESCRIPTION

A quick doughing, long working time heat-curing acrylic resin for total and partial denture bases.

Classification : Type 1 Class 1, ISO 20795-1:2013.

PROPERTIES

Various polymerization possibilities, fluid consistency, cadmium free and lasting colour.
Resistant to alcohol, fruit drinks and denture cleaners.
Can be used with any kind of acrylic or porcelain teeth.

COMPOSITION

Acrylic resin in liquid and powder form.

Powder: 98% methyl methacrylate-copolymer

Liquid: 96% methyl methacrylate-monomer

INSTRUCTIONS OF USE

WORKING TIME

Dough Time (for a pourable paste)

20 - 30 minutes, at 20 °C and with the following proportion

- . For 1 prosthesis: 24 g powder / 14 ml liquid
- . For 2 prostheses: 48 g powder / 28 ml liquid

Use the dosers supplied with the product.

Working Time

As soon as the material has doughed sufficiently and not longer sticks to the fingers, it can be worked for 40 minutes, with a room temperature of 20° C.

APPLICATION METHOD

MOULD PREPARATION

Prepare the plaster mould in a denture flask having spring closure, remove the wax by scalding it in hot water.

aint the warm mould with a separating varnish such as TECHNOSIL, taking care not to paint the teeth. Leave to dry and apply a second layer if necessary.

To ensure an appropriate bond with the denture base, routen the teeth with a bur and wet with monomer.

PREPARING THE DOUGH

- Measure out the powder and liquid in the recommended ratio
- Free measurement: Pour the liquid in a mixing receptacle, add the powder until there is a fine layer of unwetted powder on the surface, wait 30 seconds and shake the receptacle to wet all of the powder.

Mix with a spatula for 60 seconds, and ensure that no air is trapped in the mixture. The relatively fluid consistency of the mixture is misleading - there is NO need to add more powder.

Cover the receptacle and leave to rest for 20-30 minutes until the mixture gets doughed. In this point the mixture is malleable and does not stick to ones fingers.

The mixture remains plastic for a maximum of 40 minutes. (With a covered receptacle and at a temperature of 20 °C)

APPLICATION METHOD

PACKING AND CLOSING THE MOULD

Place the dough in the warm mould (about 40 °C) until there is an excess, and then cover it with a cellophane sheet. Close the mould slowly and open to remove the excess dough. Close again slowly after removing the cellophane sheet and press (200 kg/cm²) leave to rest for some 10 minutes and fix with a clamp

POLYMERISATION

a) **Standard cure:** Place the flask in cold water, heat up to 80 °C and maintain for 2 hour and then allow to boil at 100°C for a further 30 minutes.

b) **Fast cure:** Place the flask in warm water (30-40 °C). Heat up to 100 °C and leave to boil for 45 minutes.

c) **Long cure:** Place the flask in a curing device, fill with cold water and leave for 10 hours at 80 °C

After the curing process has been completed with any of the above systems, let the flask cool for at least 30 minutes exposed to the air. It may then be cooled with in cold water.

Ensure that there are at least 3 litres of water in the receptacle. Where several flasks are polymerised at the same time, there should be at least 2 litres of water per flask.

FINISHING AND POLISHING

ECO-CRYL HOT can be finished and polished using usual methods. We recommend the use of the polishing pastes POLI-R and POLI-B.

TECHNICAL INFORMATION

Residual monomer content for ECO-CRYL HOT when cured with standard method is < 0.5 % (test 8.8 ISO 20795-1:2013).

STORAGE AND EXPIRY

ECO-CRYL HOT powder and liquid retain all those qualities over a period of at least 3 years.

- Batch number and expiry date are marked on each package

- Store in a dry place at a temperature not exceeding 25 °C

- Keep out of sunlight

SAFETY INSTRUCTIONS

ECO-CRYL HOT liquid contains flammable solvents and should be handled in well-ventilated non-smoking areas, away from heat and naked flames.

DO NOT INHALE THE VAPOURS.

Contact with the skin and eyes should be avoided and suitable protective clothing should be worn, i.e. overalls, gloves and goggles. In the event of contact wash immediately with copious amounts of water. A spillage of liquid should be cleaned up immediately with sand or earth.

In the event of fire use sand, earth or a suitable extinguisher.

Store in a cool place from direct sunlight, naked flames and sparks.

Flashpoint: 10° C.

Contraindication and side effects

Do not use if the patient is known to be allergic to any of the ingredients in ECO-CRYL HOT.

In individual cases, local allergic reactions to methacrylate materials have been reported.

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EN

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet for products ECO-CRYL HOT powder and liquid. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review.

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