MODELA-CRYL

Descripción

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MODELA-CRYL

Description

Self curing modelling resin for crowns and bridges.

Indications:

- Building up patterns of outer copings with a brush
- To make the patterns of telescopic crowns, inlays, palatal and lingual bars, connectors, clasps and Maryland bridges
- Superstructures in implantology

Contraindications:

If the user is known to be sensitive to one of the constituents, it must not be used.

Properties

- Modelling resin with the brush technique.
- Fast curing (app. 4 minutes)
- Very low contraction
- Burns out without any residual, obtaining a smooth casting surface.
- Easy milling

Composition

Self curing acrylic resin based in methylmethacrylate in liquid and powder form.

Application method

- Dose the required amounts of liquid and powder into 2 different mixing vessels.
- Dip the tip of the brush first into the liquid and then into the powder, so an small resin ball is made
- Deposit the resin on the model crown and build up the secondary crown repeating the process.
- When using a gypsum model, isolate the surface previously with a separator like TECHNOSIL or TECHNOSIL SUPER.
- Repeat the procedure to obtain the complete crown.
- Once the resin has set (3-4 minutes with a temperature between 21 and 23°C) remove the pattern carefully and check the margin and the internal surface.
- Return the pattern to the model and make a uniform thickness with a bur (0.3 – 0.4 mm)

Indications

Clean the brush with the liquid MODELA-CRYL during use when necessary and also after finishing the work.
Do not pour the non used powder or liquid back to their original bottle.

Shelf life and storage

MODELA-CRYL powder and liquid retain all those qualities over a period of at least 2 years. The expiry date is shown on the container
Store in a dry place at a temperature not exceeding 25 °C.
Keep out of sunlight.

Presentation

Colour : Red
9025-100- KIT Standard: 100g + 100ml+ accessories

9025-010 Powder: 500 g
9026-010 Liquid: 250 ml

Safety cautions

MODELA-CRYL liquid contains inflammable solvents and must be used in well ventilated no smoking areas and away from naked flames and heat-generating apparatus.
Avoid contact with skin and eyes, wearing protective clothing such as gloves and glasses. In case of skin or eye contact wash liberally with water. Prolonged contact with the skin may produce hypersensitivity in susceptible persons. Stop use in case of sensitisation.
Liquid spills must be cleaned up immediately and dried. In case of fire, extinguish using a fire extinguisher, sand, or earth.
Keep in a cool dark place, away from naked flames and warm areas.
Ignition point: 10 °C.

Additional information

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions
For further information, see the safety data sheet for the products MODELA-CRYL powder and liquid, which can be requested from your usual supplier.

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