MUFLA MICROONDAS

Descripción

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MUFLA MICROONDAS - MUFFLE

Description

Special reinforced synthetic muffle for microwaves

Caution

Use a domestic microwaves of 700 – 900 watt.
A higher power could deform the muffle

This muffle is designed exclusively for microwaves.

Never use it with boiling water polymerizers or conventional polymerizing furnaces

Working Instructions

No separator between the flask and the plaster is needed

■ Setting up the flask:

Place the elements inside the MUFLA DE MICROONDAS as in conventional flasks. We recommend a special plaster MUFLA-GIPS to fill in the upper side of the flask

■ Removing the wax

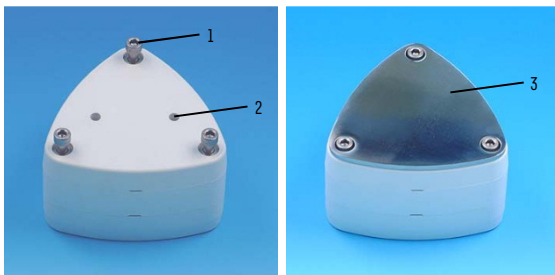
In order to achieve a perfect softening of the wax, it's enough 1'30" of radiation at highest power
An excess of radiation would cause a liquid state of the wax, which would be absorbed by the plaster, making it more difficult to eliminate

■ Pressing the flask

When the resin has got the appropriate consistence, pour it in and press it softly until 100 Kg/m²
Then carry out the suitable tests removing the excess of resin.
We recommend to press always with the **metallic protective plate (3)** which is essential to avoid damages in the flask

■ Placing of the screws

The screws (1) should be tightened with the flask under pressure, avoiding an excess of pressure and always with the protective plate (3)

1. Screws
2. Holes for the plaster escape
3. Protective plate

Curing the resin

Once placed the screws, introduce the muffle inside the microwave (without the protective plate !)

Follow the instructions of the resin's manufacturer

Guidance instructions

- for standard curing time resins (30 minutes)

Apply a maximum power radiation for 4 minutes

If the prosthesis is thicker than usual, we recommend a previous cooking at 20% pressure for 5 minutes, followed by 4 minutes at highest power

- high impact resins (3 hours curing time)

If the microwave has got 5 power levels:

- first cure at level 1 (20 %) power for 20 minutes
- then, 4 minutes at highest power (level 5)

If the microwave has got 10 power levels

- first cure at level 2 (20 %) power for 20 minutes
- then, 4 minutes at highest power (level 10)

Once the process has finished, let cool down no less than 30 minutes at room temperature

Packing

9098-000	Complete muffle with accessories
9098-100	Pressing protective plate

Dimensions

External:	10.5 x 10 x 6.8 cm
Inside:	8.7 x 7.1 x 5.5 cm
Weight:	1515 g

Additional information

The information given in this sheet is based on our level of knowledge at the moment at the date of printing and is not intended to be a warranty for the product.

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