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TECHNO SIN

Tixo

HOJA DE INSTRUCCIONES

ES

2

WORKING INSTRUCTIONS

EN

4

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IT

6

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FR

8

INFORMACIÓN ADICIONAL

10

ADDITIONAL INFORMATION

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INFORMATIONS COMPLÉMENTAIRES

CE
1639

MD

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Tixo

DESCRIPTION

- Intended purpose:** Orthodontic resin. Self curing
- Classification:** Type 1 according to ISO 20795-2:2013
- Indications:** Unproper teeth or dental arches alignment
- Intended patient population:** Patients with unproper teeth or dental arches alignment
- Medical conditions:** Patient with misaligned teeth with no contraindication
- Intended user:** Dental technician
- Intended environment:** Dental prosthetic laboratory
- Clinical benefits:** Teeth and dental arches alignment in orthodontic treatment

COMPOSITION

Acrylic resin in liquid and powder form.

Powder:	100 % Methylmethacrylate-copolymer
Liquid:	> 90 % Methylmethacrylate-monomer

INSTRUCTIONS OF USE

RECOMMENDED PRODUCT COMBINATION

General-purpose equipment habitual in the prosthetic laboratory.

APPLICATION METHOD

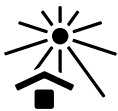
- Soak the plaster model in warm water for about 30 minutes shake the exceeding water and dry the surface with a paper handkerchief.
- Apply a layer of separating varnish as TECHNOSIL on the model and let it dry for 10 minutes.
- The usual use method is “salt and pepper” technique.
- Dispense the amount of liquid to be used in the supplied plastic pot and dose with the dropper. Do not introduce the dropper in the bottle to avoid contamination. Do not return the unused liquid to the bottle. A liquid contamination will lead to polymerization inside the bottle.
- Pour on the model 10-20 drops of liquid and turn it to achieve the dispersion of the liquid over the entire surface, creating a thin film. A brush may be used as well.
- Sprinkle the powder uniformly over the wet model. Shake the excess of powder.
Apply more layers following the same procedure to get the desired thick.
Smooth the surface with a monomer layer.
- Finish the curing in water at 40-45 °C and 2 bar of pressure for 8-10 minutes.
- To obtain pore-free plates, introduce the work into the polymeriser before 6 minutes. This time is counted from the first moment that powder and liquid come into contact.
- To get a better chemical union within the layers, we advise to start the moulding in a side of the model and put the screws or the retainers between the resin layers and coat again with liquid all the plate

TECHNO SIN Tixo can be finished and polished using usual methods. We recommend the use of POLI-R pastes for grinding and polishing and POLI-B for the final shine. Repairs can be carried out with the same material.

TECHNICAL INFORMATION

Residual monomer content for TECHNO SIN Tixo when cured with standard method is < 5 % (test 8.5 ISO 20795-2:2013).

STORAGE AND EXPIRY

TECHNO SIN Tixo powder and liquid retain all those qualities over a period of at least 2 years when stored in their original packaging, in a moisture-free environment, at a temperature not exceeding 25°C, and out of reach of sunlight.

- Batch number and expiry date are marked on each package
- Store in a dry place at a temperature not exceeding 25 °C
- The device must remain in its sealed container away from sunlight

SAFETY INSTRUCTIONS

Contient: methyl methacrylate.

Flash point: 10° C

- Highly flammable liquid and vapour.
- Causes skin irritation.
- May cause an allergic skin reaction.
- May cause respiratory irritation.
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources.
- No smoking.
- Avoid breathing vapours.
- Wear protective clothing, eye protection, face protection.
- IF ON SKIN: Wash with plenty of soap and water.
- Dispose of contents and container to an approved waste disposal plant.

Contraindication and side effects

Do not use if the patient is known to be allergic to any of the ingredients in TECHNO SIN Tixo. In individual cases, local allergic reactions to methacrylate materials have been reported.

Safe disposal of the device

For monomer and polymer, disposal should be according to regulation as stated in their SDS. For final polymerised product, no special measures are required. They are not considered hazardous waste, so they can be placed in regular waste.

ADDITIONAL INFORMATION

This device is single use. There is no risk of reuse as the material, once polymerised, cannot become powder and liquid again.

Information for dental professional to patients.

- The product, before and after polymerization (prosthesis), should not be disinfected using microwave, as it can affect the resin structure.
- Clean the product with non-aggressive and non-abrasive dental cleaning products
- Patients using the device should have periodic check-ups by a dental professional

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EN

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet for products TECHNO SIN Tixo powder and liquid. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review. (<https://brochures.protechno.com/Docs/HI/06/HI-T-TECHNO-SIN-TIXO.pdf>)

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For use only by a dental technician in the recommended indications.

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Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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