# TECHNO TRAY-P

HOJA DE INSTRUCCIONES	ES	2
WORKING INSTRUCTIONS	EN	4
ISTRUZIONI PER L'USO	IT	6
NOTICE D'EMPLOI	FR	8
INFORMACIÓN ADICIONAL ADDITIONAL INFORMATION INFORMAZIONI SUPPLEMENTARI INFORMATIONS COMPLÉMENTAIRES		10

Consulte nuestra web  
Visit our website

# TECHNO TRAY-P

## DESCRIPCIÓN

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# TECHNO TRAY-P

## DESCRIPTION

Self curing acrylic resin for individual trays and base and registration plates.

### PROPERTIES

- High dimensional stability and low curing contraction.
- Easy to handle, it doesn't stick to the fingers.
- Easy to cut
- Soft in the bur and free of powder

### APPLICATIONS

- For the production of custom trays
- For registration plates

With the colour pink:

- For the setting up of teeth in complete or partial prostheses.
- for bite rims

More rigid structures than with wax are obtained without the need of metallic reinforcement and it doesn't deform with the heat of the mouth.

Besides the colour is similar to the modelling wax

### COMPOSITION

Acrylic resin in liquid and powder form.

Powder: methylmethacrylate-copolymer

Liquid: methylmethacrylate-monomer

## INSTRUCTIONS OF USE

### APPLICATION METHOD

Measure out the powder and liquid in the ratio of:

**1 part of liquid for 4 parts of powder**

Pour the measured liquid in a mixing receptacle, add the powder and mix with a spatula for 30 – 40 seconds. The mix will stick on the spatula. Then you can pick the dough with your hands and knead it. The mixed dough must have a great plasticity

The hardening time is about **8 minutes**.

### FINISHING AND POLISHING

TECHNO TRAY-P can be finished with any kind of cut bur.

## DISPOSAL INFORMATION

Remaining stocks must be disposed of according to the corresponding national legal requirements.

## STORAGE AND EXPIRY

TECHNO TRAY-P powder and liquid retain all those qualities over a period of at least 2 years.

- Batch number and expiry date are marked on each package
- Store in a dry place at a temperature not exceeding 25 °C
- Keep out of sunlight

## SAFETY INSTRUCTIONS

TECHNO TRAY-P liquid contains flammable solvents and should be handled in well-ventilated non-smoking areas, away from heat and naked flames.

DO NOT INHALE THE VAPOURS.

Contact with the skin and eyes should be avoided and suitable protective clothing should be worn, i.e. overalls, gloves and goggles. In the event of contact wash immediately with copious amounts of water. A spillage of liquid should be cleaned up immediately with sand or earth.

In the event of fire use sand, earth or a suitable extinguisher.

Store in a cool place from direct sunlight, naked flames and sparks.

Flashpoint: 10° C.

### **Contraindication and side effects**

Do not use if the patient is known to be allergic to any of the ingredients in TECHNO TRAY-P. In individual cases, local allergic reactions to methacrylate materials have been reported.

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POLVO • POWDER • POLVERE • POUDRE

 VERDE • GREEN • VERDE • VERT

<b>9077-020</b>	TECHNO TRAY-P	(2 x 500 g)	1 kg	(2.2 lb)
<b>9077-030</b>	TECHNO TRAY-P	(8 x 500 g)	4 kg	(9 lb)

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EN

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet for products TECHNO TRAY-P powder and liquid. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review.

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FOR DENTAL USE ONLY.

In the case of serious incidents related to the product, please contact FAMADENT S.L.U. ([www.protechno.com](http://www.protechno.com)), and your responsible competent authority.

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