POLI-R

Descripción

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POLI-R

Description

Polishing gel for acrylic work with bactericide.

Uses

POLI - R is a formulated gel for polishing and shining all kinds of acrylic resins in a single operation.

It can be applied either with felt cones or with brushes.

Advantages

- High polishing speed
- Low content of ambient dust
- Contains a bactericide
- Can be diluted adding distilled water

Working instructions

Before using the product, mix it with a spatula to achieve a uniform product. While stored, the product can deposit a solid part in the bottom of the container.

Apply the gel direct onto the acrylic prosthesis and polish at a maximum speed of 1500 rpm.

If you use a polishing cone with dry rests of product, you'd better wet it before the polishing because these rests can scratch the resin.

If the product is too thick (due to water evaporation) small amounts of distilled water can be added to reach the desired consistency.

To obtain a high shine, it's recommended to finish with a cotton brush.

If any scratch still remains, you can use the product POLI-B as a final polisher.

Once ended the polishing process, wash the acrylic piece with water.

Packaging

7880-001	Pot	1 Kg
7880-M10	Test Pack	120 gr

Storage and expiry

The stability of the product is 2 years (respecting the storage conditions)

Close the package after each use.

Keep the product in his original package tightly closed and in a dry, cool place

Additional information

The instructions that appear on this sheet are based on our current knowledge and are given only as information without that implies a guarantee from Protechno.

protechno

Polígono Empordà Internacional
17469 VILAMALLA, GIRONA , SPAIN
Tel.: (34) 972 52 61 69
protechno@protechno.com
www.protechno.com