



## DIA FAST SETTING

Dust free alginate, it has a high dimensional stability and a fast setting time.

### Description / Use

Irreversible hydrocolloid with a fast setting time. Mucostatic impression material used for diagnostic models, partially edentulous arches, removable partial dentures, orthodontic appliances, preliminary impressions, antagonists and individual impression trays.

#### Standard packaging REF 3S1000

DIA Dental Impression Alginate Fast Set:  
1x453g pouch

#### Standard packaging REF 3S1002

1 DRY SAFE BOX can + 4 pouches(453g) +  
measuring cups

### Characteristics

- High gelling power, syneresis reduced
- Creamy
- Fast setting time
- Fragrance "all fruits"
- Dust Free
- Rapid water/dust absorption

### Advantages

- Higher dimensional stability
- Not compressive to the soft tissues
- Comfort for the patient
- Well tolerated also by children
- Dusts treated to not disperse microparticles
- Rapid and homogeneous mixing

### TECHNICAL DATA

Dosage ratio dust/water

Hand mixing time

Total working Time (ISO 1563)

Time in mouth

Setting time

### DIA FAST SETTING

10g/17ml

30"

1'30"/2'

1'

2'30"

The times mentioned must be intended to standard conditions of ambient temperature and water temperature at 23°C.  
Higher temperatures reduce the times, lower temperatures increase them.