



vestige

LIGHT-FAST

Vestige Light-Fast is a low viscosity impression material suitable for both the impression taking techniques: two steps (double impression) and one step (double mix). It is accurate in detecting details, it penetrates the furrow and adapts itself to the preparations by virtue of the perfect combination among fluidity, wettability and thixotropy. The material has superior mechanical properties in terms of tear resistance.

Description / Use

Light consistency vinylpolysiloxane used for the impression taking of fixed dental prosthesis, suitable for dental bridges and crowns, inlays, onlays, combined dental prosthesis and implants. It must be used together with VESTIGE PUTTY or VESTIGE TRAY MATERIAL. It can be scanned. Colour: blue. Fast setting time.

Standard packaging REF 1S2000

VESTIGE LIGHT-FAST: 2x50ml cartridges + 12 mixing tips

Standard packaging REF 1S2001

VESTIGE LIGHT-FAST ECO PACK: 4x50ml cartridges

Characteristics

Thixotropic
 Appropriate working and setting times
 Hydrophilic
 Dimensional stability
 Safe disinfection
 Easy to scan

Advantages

High fluidity without dripping
 Suitable for two-stage impression techniques
 Highly precise even with residual moisture
 Accurate and unaltered in time impressions
 Safe for dentists
 Compatible with all CAD/CAM systems

TECHNICAL DATA	VESTIGE LIGHT-FAST
ISO 4823 / ADA spec. 19	Type 3 light bodied consistency
Mixing ratio	1:1
Mixing time (23°C)	Automix
Total working time (23°C)	1'30"
Time in mouth (35°C)	2'30"
Setting time (35°C)	4'
Hardness	45±2 Shore "A"
Linear dimensional change	< 0,20%
Elastic recovery	> 99,5%