DFU

DIA VIRAGE

Alginato cromatico per impronte dentali	2
Istruzioni per l'uso	
Chromatic alginate for dental impressions	3
Instructions for use	
Alginato cromático para impresiones dentales	4
Instrucciones de uso	
Alginat chromatique pour les empreintes dentaires	5
Instructions d'utilisation	
Chromatisches Alginat für Zahnabdrücke	6
Anleitungen zur Anwendung	

TRAYART

Trayart Srl
Vía Paiette, 13/Q - 35040 Castelbaldo (PD) Italia
Tel. +39 0425 546515 www.trayart.it - info@trayart.it

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SOLO PER USO ODONTOIATRICO

Chromatic alginate for dental impressions

Instructions for use

DESCRIPTION / USE

High dimensional stability color change alginate for dental impressions of diagnostic models, edentulous arches, removable prostheses, orthodontic appliances, preliminary impressions, antagonist impressions and individual impression trays.

Warnings

Do not use in patients with known or suspected sensitivity to alginates as it may cause allergic reactions. Do not swallow, avoid contact with eyes.

Available in three versions:

3S2100 – DIA VIRAGE EXTRA FAST SET

3S2000 – DIA VIRAGE FAST SET

3S2200 – DIA VIRAGE NORMAL SET

Packaging: 453 g (1lb) bags

HOW TO USE

Open the pouch and transfer the contents to an airtight container. Shake the container to even out the volume of the product. Remove with the measuring cup and level with the spatula without compressing the alginate. Close the container immediately. Pour the powder into a flexible cup and add a dose of water to each scoop of alginate. Mix carefully following the times indicated in the table or the color variations during the work phases: purple shades during mixing, pink/orange during handling and filling the tray, yellow during the residence time and until removal from the oral cavity.

Disinfection

After cleansing, the impression can be treated with specific disinfectants for alginates. Follow the manufacturer's instructions.

Model Preparation

Impressions can be placed immediately after disinfection and are unaltered if properly stored in a sealed bag and in a suitable environment. Use specific plasters for dental technician use.

Preservation

store in a dry place at a temperature of 5-27°C (41-80°F). Materials are guaranteed until the date of expiration if stored correctly in their unopened packaging. The production batch and expiration date are indicated on the packages.

Technical data	DIA+ EXTRA FAST SET	DIA+ FAST SET	DIA+ NORMAL SET
Mixing Ratio	9 g powder / 18 ml water		
Mixing time	30 sec.		
Processing time (23°C)	1'05"	1'30"	2'
Residence time in the oral cavity (35°C)	45"	1'	1'30"
Setting time	1'50"	2'30"	3'30"

Limitation of Liability: The indications relating to TRAYART products, both oral and written, are based on the knowledge and experience acquired in the dental field. Professional users are still required to follow the instructions for use and check the suitability of the product according to the type of application. The professional user assumes all responsibility for the risks arising from improper use of TRAYART products. Any defect in the products limits the manufacturer's liability to the value of the products used by the professional. **FOR DENTAL USE ONLY**

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