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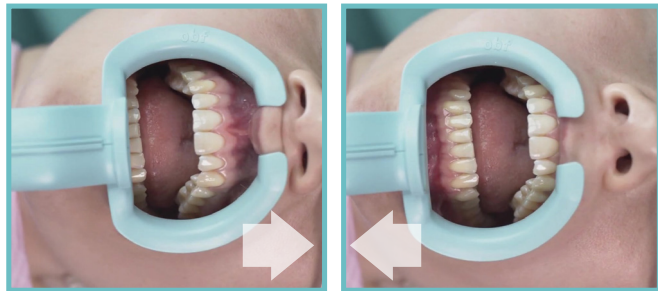
Zugang zur Studie.
Quelle NCBI: National Center for Biotechnology Information

Peer-reviewed article 2021-05-12 Original Article 1

Perioral Aerosol Sequestration Suction Device Effectively Reduces Biological Cross-Contamination in Dental Procedures

Victor Llorens¹ Maria Laura Gouveia^{2,3} Vicente Llorens-de-Lacort⁴ Maria Cristina Manzanares⁵

¹Visiting Assistant and Lecturing Staff, Department of Pathology and Biomedical Sciences, Faculty of Medicine and Health Sciences, Health University of Barcelona, Spain; ²Researching Assistant Staff, Department of Pathology, Faculty of Medicine and Health Sciences, Health University of Barcelona, Spain; ³Researching Assistant Staff, Department of Pathology, Faculty of Medicine and Health Sciences, Health University of Barcelona, Spain; ⁴Researching Assistant Staff, Department of Pathology, Faculty of Medicine and Health Sciences, Health University of Barcelona, Spain; ⁵Researching Assistant Staff, Department of Pathology, Faculty of Medicine and Health Sciences, Health University of Barcelona, Spain

Abstract
Objective: The infection risk during dental procedures is a concern because of dental professionals which has increased due to coronavirus (severe acute respiratory syndrome coronavirus 2) pandemic. The development of devices to specifically mitigate cross-contamination by droplet/aerosol is vital to stop infection transmission. The objective of this study is to assess the effectiveness of a perioral suction device (Peri Buffer, OBF) to reduce biological contamination spread during dental procedures.
Materials and Methods: Thirty patients were randomized 1:1 to a standard professional dental hygiene treatment with OBF and without. Adenosine triphosphatase (ATP) bioluminescence assay was used to evaluate the spread of potential contaminants. The total number of relative light units (RLU) from key dental operative locations (operator, hand, block, back of the surgical operator's apron, patient's safety goggles, and instrumented table) were measured. Percentage contamination reductions between control and OBF were compared.
Statistical Analysis: Primary outcome, total RLU, was analyzed by comparing the means of RLU of dental sites using a two-sided two-sample t-test. Secondary outcomes are RLU of biogel data for the different locations were analyzed in the same way. Proportion of patients from whom different locations reported events (leak, accept-able, and failed) were analyzed by using Fisher's exact test.
Results: For the whole dental environment, RLU reduction (1750 units) achieved with OBF was 98.4% (95% CI, 97.4-98.9). By dental operator location the reduction in RLU

Keywords
- dental
- ATP bioluminescent measurements
- cross-contamination
- infection disease transmission
- oral health
- COVID-2