Oral BioFilter

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Oral BioFilter

DESCRIPTION

The **obf** is a lip retractor with suction that minimises cross-contamination in the dental office setting. The **obf** is disinfected (not sterilised) prior to packaging. It comes in two separate parts (retractor and suction tube) and is packaged in an individual bag. The **obf** is made from medical-grade plastic and is for one-time use. (Do not sterilise). The retractor fits 90% of adult mouths. If it is too large for the patient’s mouth, change the curvature of the retractor by squeezing the sides and top/bottom together. During long procedures, we recommend removing the **obf** for 2-5 minutes every 20-30 minutes of dental work to give the patient a break.

METHOD OF USE

Assembly: Open the bag only when you are ready to use the retractor. Assemble the two parts (retractor and tube) as shown in **Figure No. 1**. Once assembled, the tube cannot be removed.

Placing the **obf in the patient’s mouth:** Once the parts have been assembled, place the **obf** in the patient’s mouth, as shown in **Figure No. 2**. Insert one wing first and then gently bend the **obf** to insert the other wing into the patient’s mouth, as shown in **Figure No. 3**. Once in place, the wings will retract the patient’s cheeks and keep the patient’s mouth open. Guide the patient’s lips into the groove around the edge of the retractor.

Connecting the suction system: Connect the **obf** suction tube to the surgical suction system, making sure the connection is tight, as shown in **Figure No. 4**. The **obf** suction tube has an internal diameter of 16 mm. If you need an internal diameter of 11 mm, use the connection adapter. See **Figure No. 5**. The connection adapter is an accessory and is supplied separately.

Removing the **obf from the patient’s mouth:** Remove the **obf** from the patient’s mouth in reverse order, by first gently removing one wing of the retractor and then the other.

Recycling: The material used to make the **obf** can be recycled. After using the single-use device, dispose of the **obf** with the relevant recycling waste.

STORAGE AND SHELF LIFE

Store the **obf** in a cool place, away from direct sunlight. The **obf** does not have a shelf life, provided that it is stored properly. However, it is recommended that the **obf** be used within 5 years of the date of manufacture.

OTHER INFORMATION

This device has been manufactured for dental use and must be handled according to the instructions for use. The manufacturer is not responsible for any injury or damage resulting from misuse or mishandling. The user is also responsible for checking that the device is in good condition, as indicated in this Instruction Manual, before use. Refer to the latest version of the Instruction Manual on our website as this information is constantly revised.

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